



TAKING CARE OF MY SURESTEP SMOs

We are confident that with regular use, SureStep can become an amazing tool in your child's gross motor development. The changes that are overwhelmingly seen are more rapid progression through gross motor development, improved balance and coordination and increased endurance. SureStep can only be effective when it is properly fit and when it is properly secured on the foot.

REDNESS

Some redness is okay. It is very normal to have some slight redness around edges if the SureSteps are properly worn. Severe redness should be brought to the attention of your orthotist.

TIGHTNESS

Tightness is crucial to proper functioning of the SureStep system. If your child will be staying with other caregivers, please instruct them on proper fit and securing of the SureStep system.

GROWTH

SureStep must properly fit in order to correctly work. On average, children will outgrow their SureSteps in about one (1) year. Some children may outgrow them sooner than this, but often children can go much longer as well.

SHOES & SANDALS

SureStep makes both an athletic shoe and a sandal that are designed specifically for children who wear orthoses. Both styles provide additional width, depth and flexibility to allow for a comfortable fit. Our shoes and sandals are available to purchase through an orthotist or online at surestep.net.

CARE

Mild soap and water may be used to clean any soiled areas. Allow to air dry.